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ORAL DOSE SYRINGE AND ADAPTER FOR ORION PHARMA'S ORAL PREPARATIONS

The oral dose syringe is intended for the dosing of liquids including liquid medicines. The adapter is an accessory that is intended

to be used together with oral dose syringes for the administration of liquid oral preparations of Orion Pharma. The role of the adapter is to ease the administration process. The adapter assists in enabling a safe, accurate and easy drug administration for the patient together with the oral dose syringe.

Instructions for use:

1. Open the cap of the bottle and attach the adapter to the bottle.
2. Insert the tip of the dosing syringe into the adapter hole.
3. Turn the bottle upside down and pull the plunger to get the instructed amount of medicine into the syringe.
4. Turn the bottle into an upright position and remove the syringe.
5. Administer the medicine into the patient's mouth or dispense it on a spoon.
6. Remove the adapter and close the bottle with its original cap. Store the package according to instructions. **Do not use the adapter as a bottle cap during storage.**
7. **Rinse the syringe and the adapter immediately after use.** Pull the plunger completely out and wash the syringe, plunger and adapter separately under cold running water. **Do not use hot water or detergents.**
8. **Do not use the syringe after the expiration date presented as YYMMDD printed on the pack next to the  symbol.**

To report any serious incident, contact the manufacturer and the Competent Authority (MHRA, tel. 020 3080 6000) of the Member State in which the user is established. This is important for the on-going monitoring of the benefit/risk balance of the product.

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Orion Corporation
elm-plastic GmbH